

# A Walk Around Tour of the HaloFogger®

**Rear Handle**

**Status Indicator Light**

**High Precision Nozzle**

Dry-mist of disinfectant is dispensed through nozzle.

**Digital Timer**

Set the timer from the instructions on the rear of device

**Start Button**

Press and hold for 2 seconds to begin unit's countdown and the treatment process.

**Disinfectant Fluid Level Indicator**

Shows how much disinfectant is in the reservoir and when it is time to refill.

**Reservoir Door**

Push and lift open to expose funnel port for refilling.

**Front Handle**

**Reservoir Door**

**Funnel Port**

**Large Wheels**

For easy maneuvering.

**Power Cord Connection**

For more information,  
please contact:



800-424-2436  
[www.quiplabs.com](http://www.quiplabs.com)



**AUTHORIZED RESELLER**  
Surface Disinfection Products